Levitist (C. albicans)	EN 13624, EN 17387, EN 13697, EN 14562	5
Tuberkuuloit (M. terreae)	EN 14348, EN 14563	5
Zarflu viirusele karšu viirust (HIV, HBV, HCV dahil)	EN 14476, EN 16777	3

* testi S. aureus, P. aeruginosa, E. hirae, E. coli üleindele geireklettitudi

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SOLO PER USO PROFESSIONALE

EN - INSTRUCTIONS FOR USE: Disinfectant and cleaner for medical device surfaces.

PACKS: 2500 ml container

ACTIVE INGREDIENTS: 100 g of Zeta 3 Soft and Soft Classic contain 34.4 g ethanol, 14 g isopropanol, non-ionic surfactants, additives, auxiliaries and water to 100 g.

CONTRAINDICATIONS: No known contraindications.

WARNINGS: We always recommend testing the product on a small area before proceeding to treat the entire surface. Flammable liquid and vapours. Causes serious eye irritation. In the event of contact with eyes, rinse thoroughly for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. If the eye irritation persists, consult a doctor.

Contains: (R)-P-MENTA-1,8-DIEN. May cause an allergic reaction. If irritation, reddening or other signs of hypersensitivity occur during use, stop using and consult a doctor.

PRECAUTIONS: Wear gloves and protect the eyes and face. Wash the hands thoroughly after use. Keep away from sources of heat, sparks, naked flames and hot surfaces. Do not smoke. Keep in a tightly-closed container. Do not dispose of the product in the environment; dispose in accordance with local laws and regulations.

SIDE EFFECTS: No side effects have been noted.

STEP-BY-STEP INSTRUCTIONS: Spray directly on the surface to be disinfected. Wipe the entire surface to be disinfected thoroughly before the product has evaporated completely, then leave to dry.

SPECTRUM OF ACTION

Effectiveness (microorganisms)	Reference standard <i>Tests carried out in dirty conditions</i>	Contact time (minutes)
Bactericidal (S. aureus, P. aeruginosa, E. hirae)	EN 13727, EN 17387, EN 13697*, EN 14561	5
Yeasticidal (C. albicans)	EN 13624, EN 17387, EN 13697, EN 14562	5
Tuberculocidal (M. terreae)	EN 14348, EN 14563	5
Virucidal against enveloped viruses (Inclusive HIV, HBV, HCV)	EN 14476, EN 16777	3

*tested on S. aureus, P. aeruginosa, E. hirae, E. coli

IMPORTANT NOTES: The instructions must be observed, irrespective of possible information given in any other way, including during demonstrations. It is the operator's responsibility to ensure that the product is suitable for the application in question. The Company is not liable for possible damage, including to third parties, deriving from failure to observe the instructions or inappropriate use of the product and in any case within the limits of the value of the products supplied.

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