EN - INHALE: Flammable liquid and vapour. Causes serious eye irritation. Contains: (R)-p-mentha-1,8-diene. May produce an allergic reaction. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking, keep container tightly closed. Wash hands thoroughly after handling. Wear protective gloves / eye protection / face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.

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specific for surfaces

aldehydes and phenols free

- Bactericidal
- Yeasticidal
- Tuberculocidal
- Virucidal vs Enveloped viruses (HIV, HBV, HCV included)

120 wipes

disinfectant for medical devices | www.zhermack.com

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SOLO PER USO PROFESSIONALE

EN - INDICATIONS FOR USE: Wipes dampened with an alcoholic disinfectant solution and detergent for rapid disinfection of small medical device surfaces.

Packaging: tub of 120 wipes (13 x 19 cm).

ACTIVE INGREDIENTS: 100 g disinfectant solution containing 34.4 g ethanol, 14 g isopropanol, excipients and fragrance.

CONTRAINDICATIONS: No known contraindications.

WARNINGS: It is advisable to always try the product on a limited area before proceeding across the entire surface. Use wipes within 30 days of opening the package. Do not reuse wipes as doing so could result in loss of efficacy and cross-contamination. Flammable liquid and vapours. Causes serious eye irritation. In case of contact with eyes, rinse thoroughly for several minutes. Remove any contact lenses if easy to do so. Continue rinsing. Consult a doctor if eye irritation persists. Contains: (R)-P-MINT-1,8-DIENE. May cause an allergic reaction. Stop using and consult a doctor if irritation, redness or other signs of hypersensitivity occurs during use.

PRECAUTIONS: Wear gloves and eye and face protection. Wash hands thoroughly after use. Keep away from heat, sparks, open flames and hot surfaces. Do not smoke. Keep container tightly closed. Do not litter. Dispose of product according to local regulations.

ADVERSE REACTIONS: No adverse reactions have been found.

STEP-BY-STEP INSTRUCTIONS: Follow the above illustrations when first opening the tub. Thoroughly scrub all surfaces to be disinfected and let dry.

RANGE OF ACTION:

Effectiveness (microorganisms)	Reference standard* - Tests carried out in dirty conditions	Contact time (minutes)
Bactericidal (S. aureus, P. aeruginosa, E. hirae)	EN 13727, EN 17387, EN 13697**, EN 14561	5
Yeasticidal (C. albicans)	EN 13624, EN 17387, EN 13697, EN 14562	5
Tuberculocidal (M. terre)	EN 14348, EN 14563	5
Virucidal against enveloped viruses (including HIV, HBV, HCV)	EN 14476, EN 16777	3

*Standards related to the solution. Tests carried out in dirty conditions. **tested on S. aureus, P. aeruginosa, E. hirae, E. coli

Abatement effectiveness of wipes (Method prEN 16615, 10' contact): R->4log vs. S.aureus, E.hirae. R->3log vs. Calbicans, Paeruginosa.

IMPORTANT REMARKS: Any information provided in any way, including during the demonstrations, does not replace the instructions for use. The operator is responsible for ensuring that the product is suitable for its expected application. The Company shall not be held liable for damages, including those of third parties, which are the consequence of a failure to follow instructions or unsuitability for application and, in any case, within the value limits of the products supplied.

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